




How to activate
the offline mode

In order to successfully complete the steps detailed in this document you should, at least, have done the following:

1. Successfully installed and registered the corresponding treatment station.
2. Ensure that the treatment station has successfully synchronized with the server at least **once in the past 7 days.**

STEP 1. Start the Multi-User Treatment Station by clicking on the Rehametrics icon on your desktop.

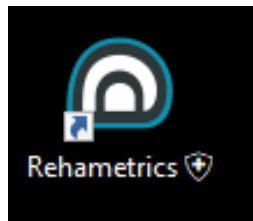

STEP 2. After starting the software, a screen similar to the one shown below will appear.




There are two options to continue without an Internet connection:

1- Wait 120-150 seconds.

2- Click on the "I can't right now" button. This will take you to STEP 3 immediately.

I can't right now

STEP 3. After following either one of the options listed in STEP 2, you should access the patient list. From now on, you should be able to use Rehametrics without an Internet connection.

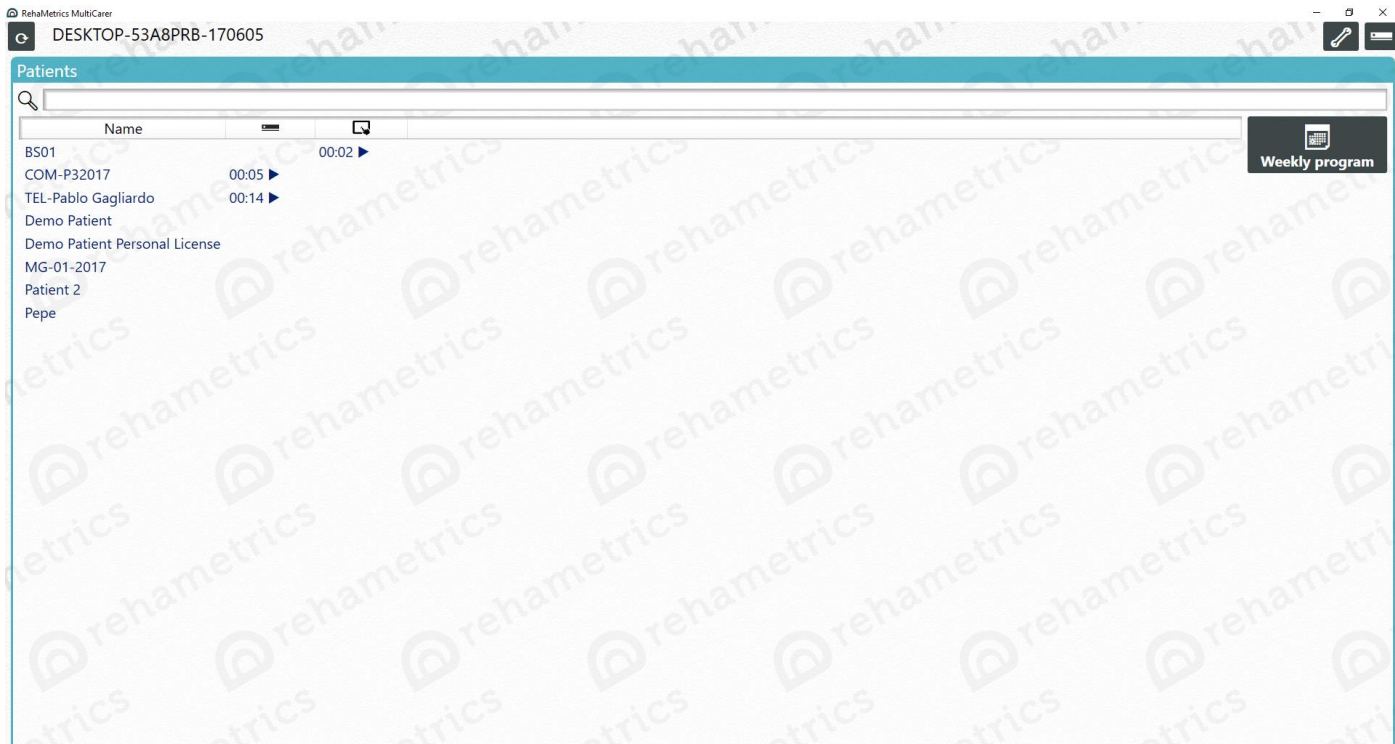

STEP 4. Please repeat STEP 2 again, if a similar screen appears.

STEP 5. Remember that, while using Rehametrics Online without an Internet connection, you will not be able to make changes to the patient's treatment plan nor access the data collected during the completed sessions. You will first need to re-synchronize the software with the server.

To do this, you need to follow these steps:

STEP 5A. Close the Rehametrics Online Multi-User Treatment Station software.

STEP 5B. Make sure that your device is now connected to the Internet.

STEP 5C. Restart the Rehametrics Online Multi-User Treatment Station software.



www.rehametrics.com