



# How to assign a patient to a Personal Treatment Station

Before following the steps detailed in this document, make sure that:

1. The patient you want to assign to the Personal Treatment Station already exists in your Online Clinic. Patients should also be currently enabled.
2. Rehametrics has been correctly installed and registered in the device that will be used as the Personal Treatment Station. Make sure the station is currently enabled.



**STEP 1.** Make sure that the device is connected to the Internet.

**STEP 2.** Log in to the software by opening the Clinic Manager Azure application.

Enter your server domain address, clinic name, user name and password to access your Online Clinic.


**NOTE:** In order to assign a Personal Treatment Station to a patient, you will have to access your Online Clinic using an account that has an administrator or technical user profile.

**STEP 3.** Once inside the Main Menu, click on the "Platforms" button.

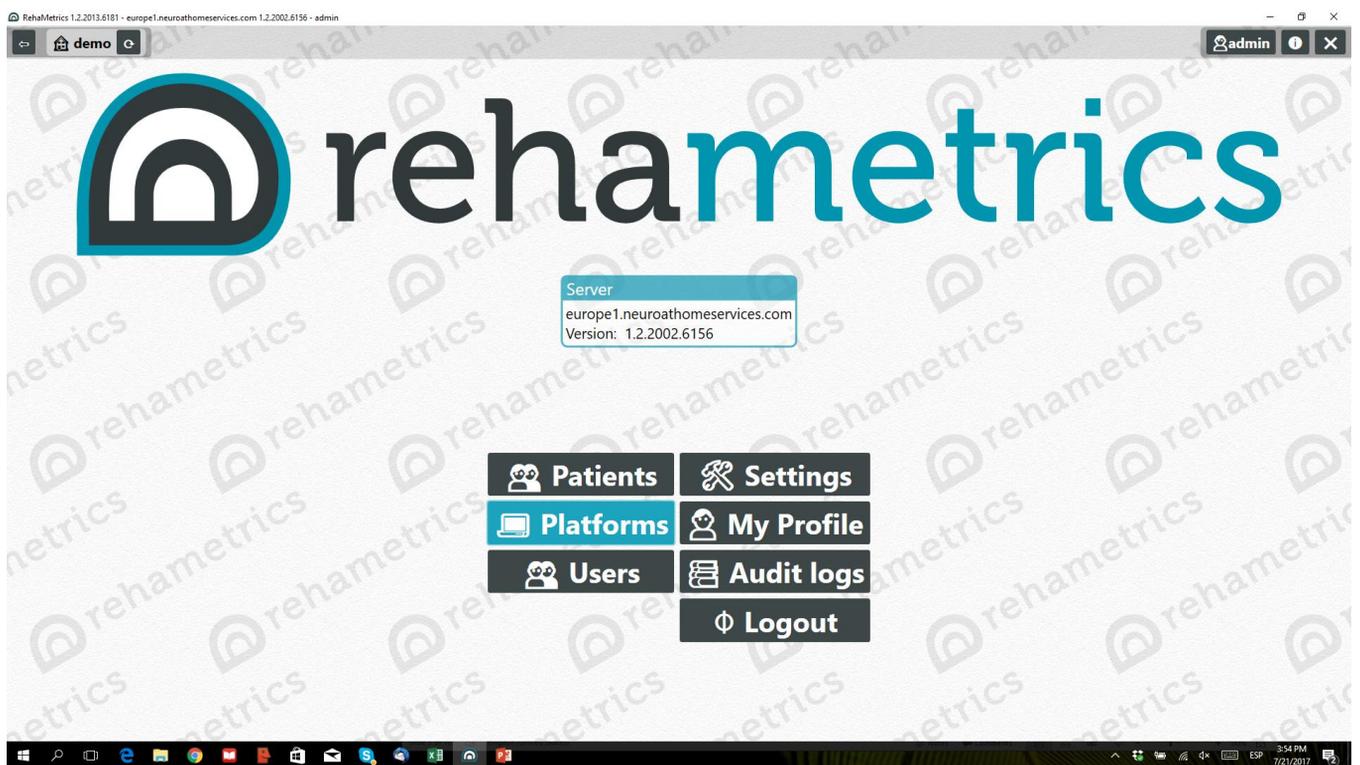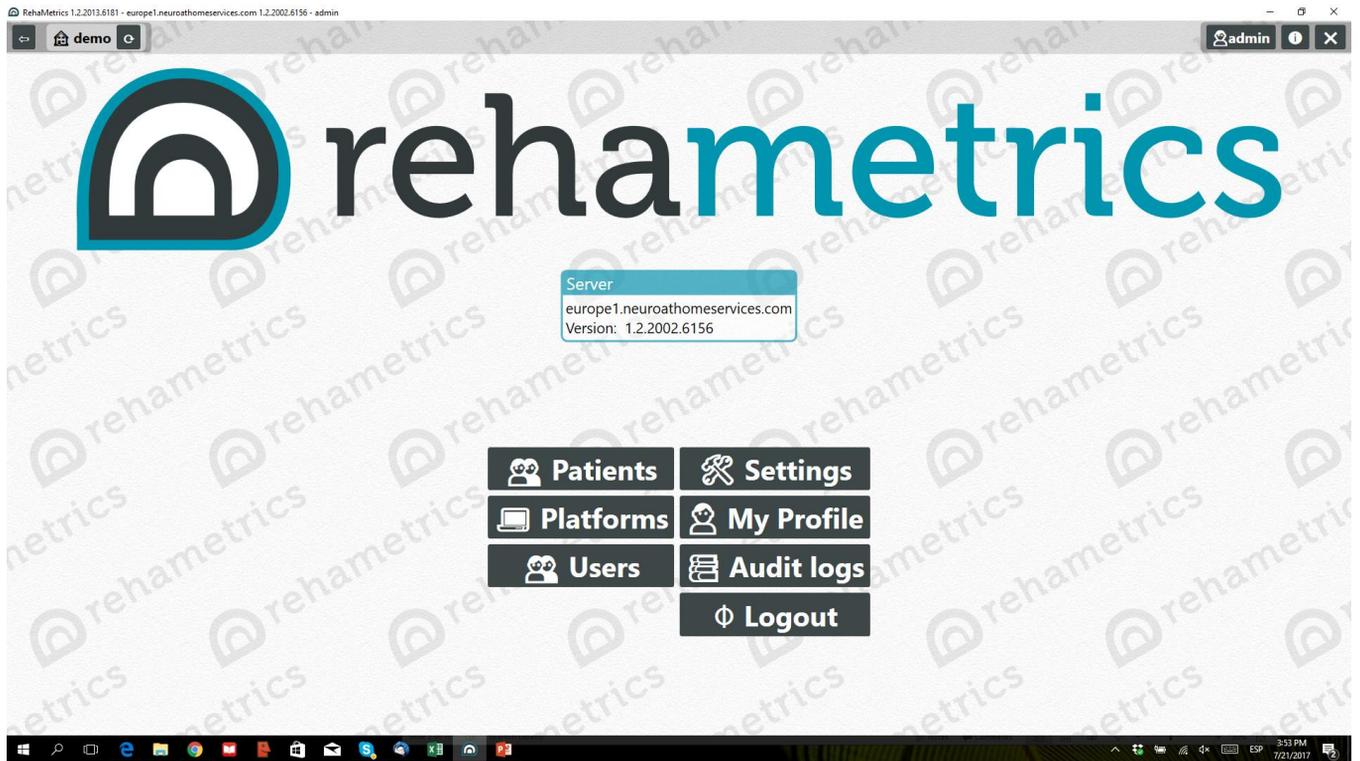

**STEP 4.** Select the device you want to become the Personal Treatment Station from the list shown. It should have the name assigned during the registration process.

**NOTE:** As you can see in the image below, there are no patients assigned to the selected platform. Notice that there is nothing displayed -for the selected platform- under the third column, "Bound Patients".

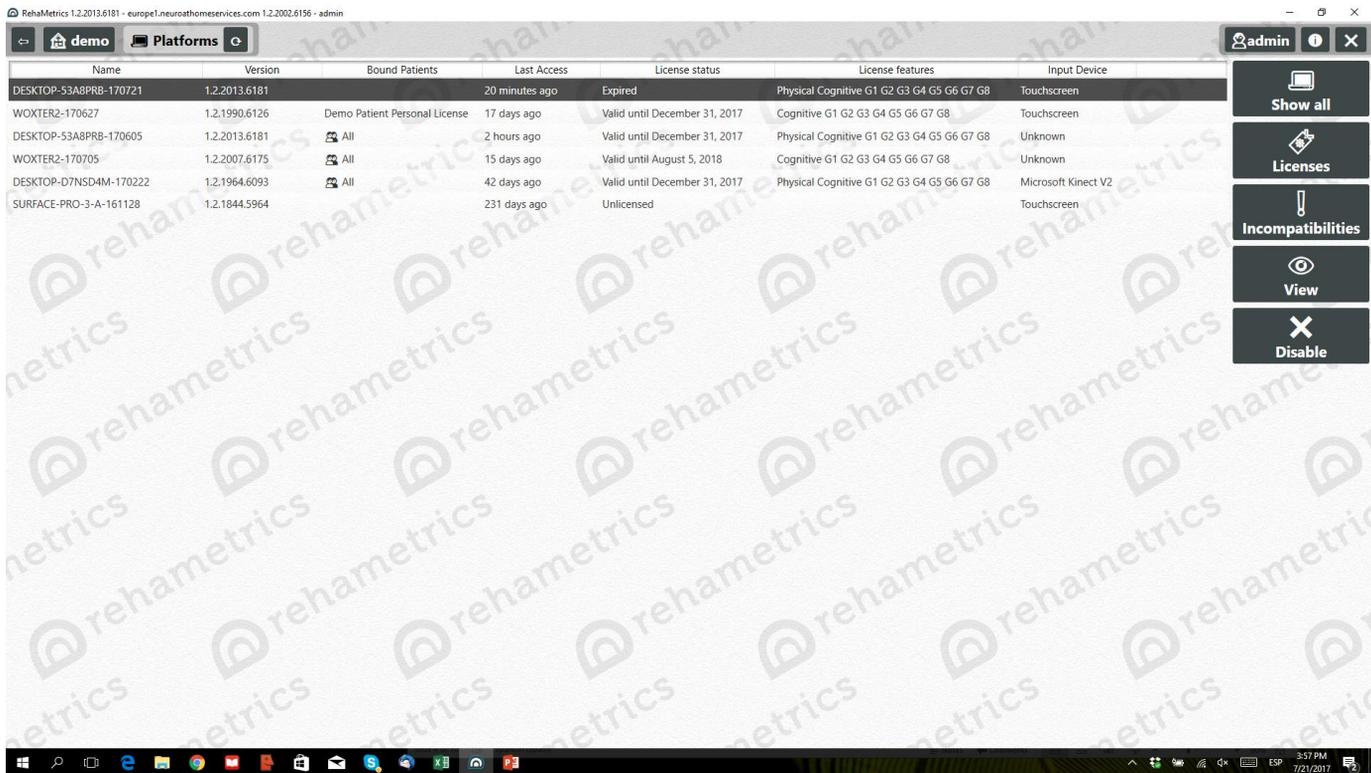

The screenshot shows the RehaMetrics admin interface. The main content is a table with the following columns: Name, Version, Bound Patients, Last Access, License status, License features, and Input Device. The table lists five devices. The 'Bound Patients' column for all devices is empty, indicating no patients are assigned to any of the listed platforms.

Name	Version	Bound Patients	Last Access	License status	License features	Input Device
DESKTOP-53A8PRB-170721	1.2.2013.6181		20 minutes ago	Expired	Physical Cognitive G1 G2 G3 G4 G5 G6 G7 G8	Touchscreen
WOXTER2-170627	1.2.1990.6126	Demo Patient Personal License	17 days ago	Valid until December 31, 2017	Cognitive G1 G2 G3 G4 G5 G6 G7 G8	Touchscreen
DESKTOP-53A8PRB-170605	1.2.2013.6181	All	2 hours ago	Valid until December 31, 2017	Physical Cognitive G1 G2 G3 G4 G5 G6 G7 G8	Unknown
WOXTER2-170705	1.2.2007.6175	All	15 days ago	Valid until August 5, 2018	Cognitive G1 G2 G3 G4 G5 G6 G7 G8	Unknown
DESKTOP-D7NSD4M-170222	1.2.1964.6093	All	42 days ago	Valid until December 31, 2017	Physical Cognitive G1 G2 G3 G4 G5 G6 G7 G8	Microsoft Kinect V2
SURFACE-PRO-3-A-161128	1.2.1844.5964		231 days ago	Unlicensed		Touchscreen

The interface also includes a sidebar on the right with buttons for 'Show all', 'Licenses', 'Incompatibilities', 'View', and 'Disable'. The top navigation bar shows 'demo' and 'Platforms'. The bottom of the screenshot shows the Windows taskbar with the time 3:57 PM on 7/21/2017.

## STEP 5. Click on the "View" button.

RehaMetrics 1.2.2013.6181 - europe1.neurothomeservices.com 1.2.2002.6156 - admin

demo Platforms



Name	Version	Bound Patients	Last Access	License status	License features	Input Device
DESKTOP-53A8PRB-170721	1.2.2013.6181		20 minutes ago	Expired	Physical Cognitive G1 G2 G3 G4 G5 G6 G7 G8	Touchscreen
WOXTER2-170627	1.2.1990.6126	Demo Patient Personal License	17 days ago	Valid until December 31, 2017	Cognitive G1 G2 G3 G4 G5 G6 G7 G8	Touchscreen
DESKTOP-53A8PRB-170605	1.2.2013.6181	All	2 hours ago	Valid until December 31, 2017	Physical Cognitive G1 G2 G3 G4 G5 G6 G7 G8	Unknown
WOXTER2-170705	1.2.2007.6175	All	15 days ago	Valid until August 5, 2018	Cognitive G1 G2 G3 G4 G5 G6 G7 G8	Unknown
DESKTOP-D7NSD4M-170222	1.2.1964.6093	All	42 days ago	Valid until December 31, 2017	Physical Cognitive G1 G2 G3 G4 G5 G6 G7 G8	Microsoft Kinect V2
SURFACE-PRO-3-A-161128	1.2.1844.5964		231 days ago	Unlicensed		Touchscreen

admin

- Show all
- Licenses
- Incompatibilities
- View**
- Disable

4:00 PM 7/21/2017

**STEP 6.** Select the patient that you want to assign to the Personal Treatment Station. Click on the top arrow (the one pointing to the right) to assign the patient. The patient has now been assigned to the Personal Treatment Station.

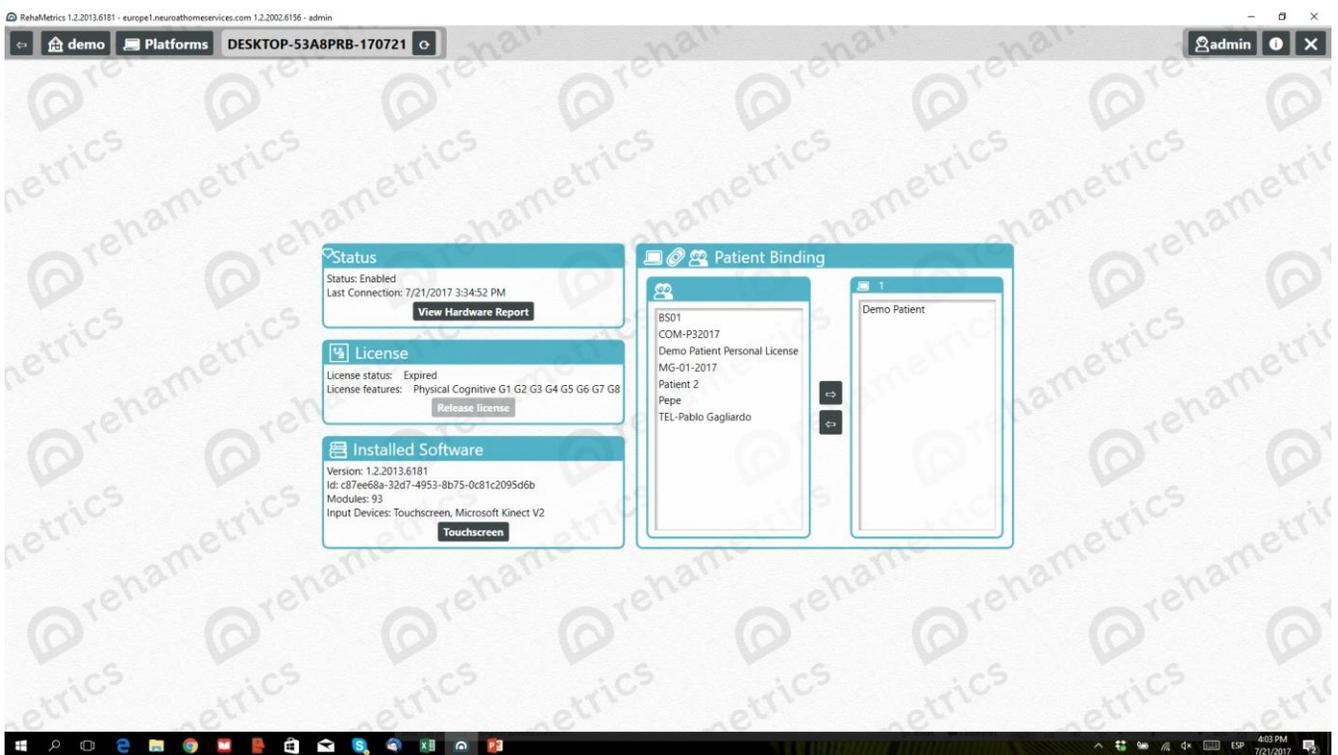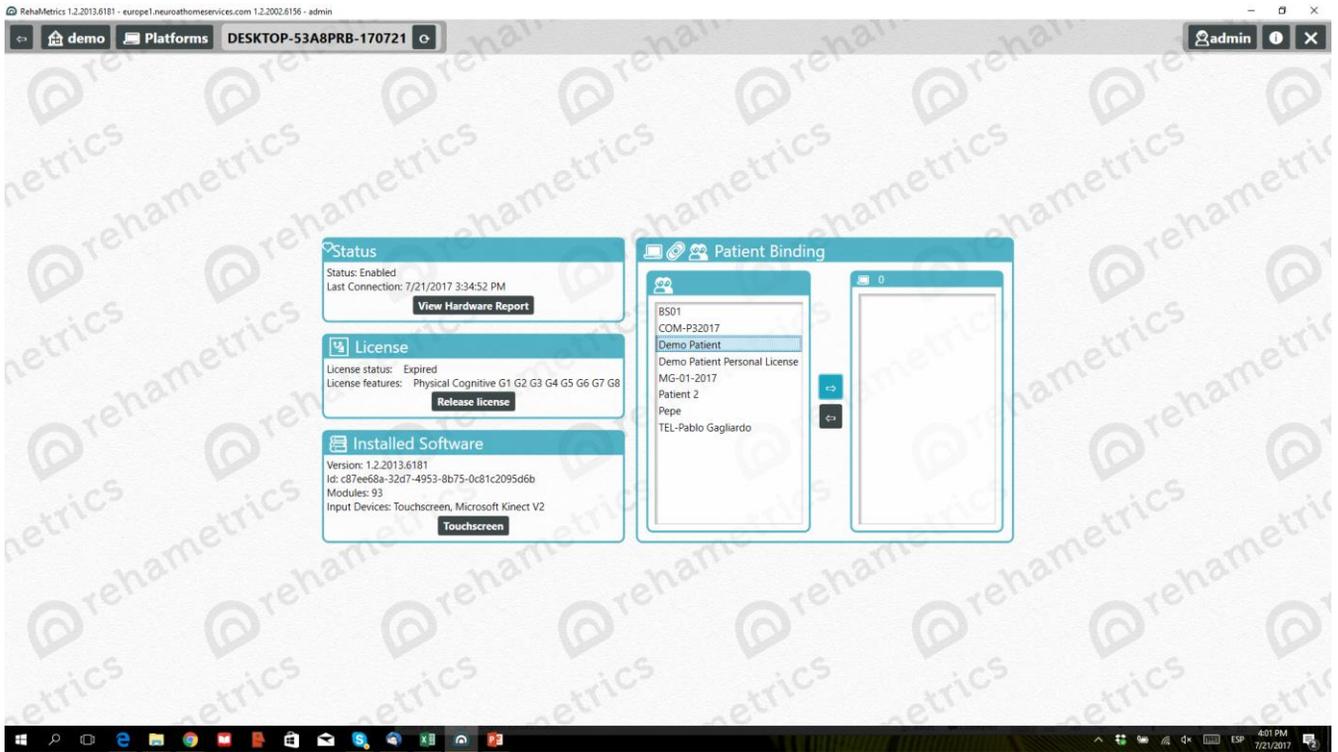

**STEP 7.** Now, when you go back to the list of available platforms, you should see that the patient now appears under the "Bound Patients" column for the selected platform.

The screenshot displays the RehaMetrics software interface. The browser address bar shows the URL: `RehaMetrics 1.2.2013.6181 - europe1.neuroathomeservices.com 1.2.2002.6196 - admin`. The page title is "demo" and the current view is "Platforms". The main content is a table with the following columns: Name, Version, Bound Patients, Last Access, License status, License features, and Input Device. The table contains five rows of data. On the right side, there are three buttons: "Show all", "Licenses", and "Incompatibilities". The Windows taskbar at the bottom shows the time as 4:04 PM on 7/21/2017.

Name	Version	Bound Patients	Last Access	License status	License features	Input Device
DESKTOP-53A8PRB-170721	1.2.2013.6181	Demo Patient	29 minutes ago	Expired	Physical Cognitive G1 G2 G3 G4 G5 G6 G7 G8	Touchscreen
WOXTER2-170627	1.2.1990.6126	Demo Patient Personal License	17 days ago	Valid until December 31, 2017	Cognitive G1 G2 G3 G4 G5 G6 G7 G8	Touchscreen
DESKTOP-53A8PRB-170605	1.2.2013.6181	All	3 hours ago	Valid until December 31, 2017	Physical Cognitive G1 G2 G3 G4 G5 G6 G7 G8	Unknown
WOXTER2-170705	1.2.2007.6175	All	15 days ago	Valid until August 5, 2018	Cognitive G1 G2 G3 G4 G5 G6 G7 G8	Unknown
DESKTOP-D77NSD4M-170222	1.2.1964.6093	All	42 days ago	Valid until December 31, 2017	Physical Cognitive G1 G2 G3 G4 G5 G6 G7 G8	Microsoft Kinect V2
SURFACE-PRO-3-A-161128	1.2.1844.5964		231 days ago	Unlicensed		Touchscreen

**STEP 8.** Go back to the Main Menu and click on the "Patients" button.

Now, when selecting the patient you have just assigned to the Personal Treatment Station, a new button called "Edit Home Plan" will appear. By clicking on this button, you will be able to design weekly treatment plans for your patient.

The screenshot displays the RehaMetrics software interface. At the top, there is a navigation bar with a 'demo' button and a 'Patients' button. Below this is a table listing patients and their treatment progress. The table has columns for 'Name', 'Platform', 'Previous Week Results', 'Current Week Results', and 'Warnings'. The 'Demo Patient' row is highlighted in dark grey. The 'Edit Home Plan' button is highlighted in blue in the right-hand sidebar.

Name	Platform	Previous Week Results	Current Week Results	Warnings
BS01		Sunday: 0% Monday: X Tuesday: 0% Wednesday: ? Thursday: ? Friday: ? Saturday: ?	Sunday: ? Monday: ? Tuesday: ? Wednesday: ? Thursday: ? Friday: ? Saturday: X	
COM-P32017		Sunday: 100% Monday: 75% Tuesday: 0% Wednesday: ? Thursday: X Friday: X Saturday: X	Sunday: X Monday: X Tuesday: X Wednesday: X Thursday: X Friday: X Saturday: X	
Demo Patient	Today	Sunday: ? Monday: ? Tuesday: ? Wednesday: ? Thursday: ? Friday: ? Saturday: ?	Sunday: ? Monday: ? Tuesday: ? Wednesday: ? Thursday: ? Friday: ? Saturday: ?	
Demo Patient Personal License	17 days ago	Sunday: ? Monday: ? Tuesday: ? Wednesday: ? Thursday: ? Friday: ? Saturday: ?	Sunday: ? Monday: ? Tuesday: ? Wednesday: ? Thursday: ? Friday: ? Saturday: X	
MG-01-2017		Sunday: 100% Monday: 75% Tuesday: 0% Wednesday: ? Thursday: ? Friday: ? Saturday: ?	Sunday: ? Monday: ? Tuesday: ? Wednesday: ? Thursday: ? Friday: ? Saturday: X	
Patient 2		Sunday: ? Monday: ? Tuesday: ? Wednesday: ? Thursday: ? Friday: ? Saturday: ?	Sunday: ? Monday: ? Tuesday: ? Wednesday: ? Thursday: ? Friday: ? Saturday: X	
Pepe		Sunday: X Monday: X Tuesday: X Wednesday: 0% Thursday: ? Friday: ? Saturday: ?	Sunday: ? Monday: ? Tuesday: ? Wednesday: ? Thursday: ? Friday: ? Saturday: X	
TEL-Pablo Gagliardo		Sunday: X Monday: 0% Tuesday: 0% Wednesday: 0% Thursday: X Friday: X Saturday: X	Sunday: X Monday: X Tuesday: X Wednesday: X Thursday: X Friday: X Saturday: X	



[www.rehametrics.com](http://www.rehametrics.com)